

SEQUENCE FOR ADVANCED DOFFING PERSONAL PROTECTIVE EQUIPMENT (PPE)

REMEMBER! The outside and front of your PPE, including the sleeves of the gown, are contaminated. If hands become visibly contaminated during PPE removal, wash hands before continuing.

1. REMOVE GOWN THEN GLOVES

Outside of gown and gloves are **CONTAMINATED!**

1a.

- With gloved hands, grasp the front of the gown at the shoulders and pull the gown forward to snap the tabs at the back
- Peel off the gown, rolling it inside out

1b.

- Continue over the gloves so that they too are rolled inside out
- Discard gown and gloves into a waste container
- Perform hand hygiene using ABHR or soap and water



2. REMOVE PROTECTIVE EYEWEAR

- Grasp the ear or head piece and lift away from face
- Place reusable eyewear in the designated receptacle for cleaning or
- Discard disposable eyewear into waste container for disposal



3. REMOVE MASK OR PARTICULATE RESPIRATOR (N95 OR P2)

- Remove by touching ties or elastics only, start with the bottom then top ties or bands
- Discard in designated waste container



4. PERFORM HAND HYGIENE

- Wash hands with soap and water or use an alcohol-based hand rub immediately after removing PPE



www.halyardhealth.com.au