

SEQUENCE FOR DONNING PERSONAL PROTECTIVE EQUIPMENT (PPE):

1. PROTECT YOUR BODY

Ensure the gown:

- Covers torso from neck to knees
- Covers arms to end of wrists
- Wraps around the sides to fasten at neck and back



2. PROTECT YOUR FACE

- Create a curve in the flexible band for the nose bridge and expand the mask
- Secure the ties or elastics at the middle of the head and neck, or fasten the ear loops
- Fit it snugly to your face and below your chin – fit check the N95 and P2 mask
- Place the face shield or eyewear over the face and eyes and adjust to fit




3. PROTECT YOUR HANDS

- Extend the cuffs to cover the wrist of the gown
- Limit the surfaces touched
- Change gloves when torn or heavily contaminated
- Perform hand hygiene as required



FINALLY
STOP and
CHECK all PPE
is in place

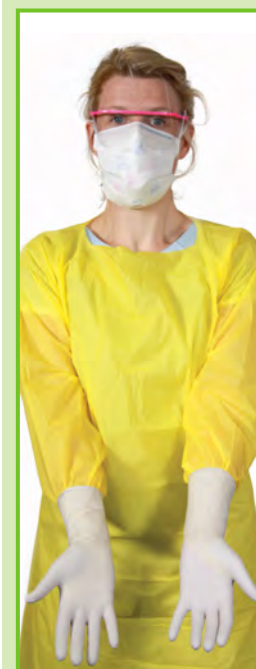

SEQUENCE FOR ADVANCED DOFFING PERSONAL PROTECTIVE EQUIPMENT (PPE):

Remember! The outside and front of your PPE, including the sleeves of the gown, are contaminated. If hands become visibly contaminated during PPE removal, wash hands before continuing.

REMOVE GOWN THEN GLOVES

Outside of gown and gloves are CONTAMINATED!

1a.

- With gloved hands, grasp the front of the gown at the shoulders and pull the gown forward to snap the tabs at the back
- Peel off the gown, rolling it inside out


1b.

- Continue over the gloves so that they too are rolled inside out
- Discard gown and gloves into a waste container
- Perform hand hygiene using ABHR or soap and water



2. REMOVE PROTECTIVE EYEWEAR

- Grasp the ear or head piece and lift away from face
- Place reusable eyewear in the designated receptacle for cleaning or
- Discard disposable eyewear into waste container for disposal

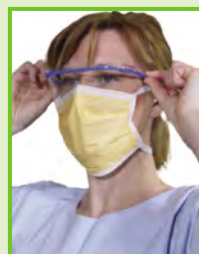

3. REMOVE MASK OR PARTICULATE RESPIRATOR (N95 OR P2)

- Remove by touching ties or elastics only, starting with the bottom then top ties or bands
- Discard in designated waste container



4. PERFORM HAND HYGIENE

- Wash hands with soap and water or use an alcohol-based hand rub immediately after removing PPE



PROTECTIVE APPAREL



HALYARD* ISOLATION GOWNS

- For use when expected risk of exposure to fluid is between low and moderate
- Medium-weight tri-layer fabric
- Tape-tab neck closure
- Tie waist, elastic cuffs

Code	Description	Packaging
69979	Isolation Gown, Yellow, Large	Eaches/Bundle = 10 Bundles/Case = 10 Eaches/Case = 100
69988	Isolation Gown, Yellow, X-Large	Eaches/Bundle = 10 Bundles/Case = 10 Eaches/Case = 100

NOTE: Please see package inserts for directions for use for each of these products
Always read the label and follow the directions for use..

PROTECTIVE APPAREL

HALYARD* IMPERVIOUS COMFORT GOWNS



69455

- Potential for moderate to heavy fluid contact in front
- Tri-layer spunbond with polyethylene laminate coating
- Breathable spunbond back
- Knit cuffs
- Velcro neck tabs
- Tie waist

Code	Description	Packaging
69455	Impervious Open Back Gown, Knit Cuff, Blue, Large	Eaches/Bundle = 30 Bundles/Case = 2 Eaches/Case = 60
5511	Impervious Full Back Comfort Gown, Yellow, Regular	Eaches/Case = 50



5511

5512	Impervious Full Back Comfort Gown, Yellow, X-Large	Eaches/Case = 50
------	----------------------------------------------------	------------------

IMPERVIOUS THUMBS-UP* FILM GOWNS

- Potential for soaking and heavy fluid contact in front
- Impervious polyethylene film
- Open back, thumbhooks



Code	Description	Packaging
7000	Impervious Thumbs-Up* Film Gown with Thumbhooks, Blue, Regular	Eaches/Dispenser = 15 Dispensers/Case = 5 Eaches/Case = 75
7001	Impervious Thumbs-Up* Film Gown with Thumbhooks, Blue, X-Large	Eaches/Dispenser = 15 Dispensers/Case = 5 Eaches/Case = 75
7002	Impervious Thumbs-Up* Film Gown with Thumbhooks, Yellow, Regular	Eaches/Dispenser = 15 Dispensers/Case = 5 Eaches/Case = 75
7003	Impervious Thumbs-Up* Film Gown with Thumbhooks, Yellow, X-Large	Eaches/Dispenser = 15 Dispensers/Case = 5 Eaches/Case = 75

NOTE: Please see package inserts for directions for use for each of these products. Always read the label and follow the directions for use.

PROTECTIVE APPAREL



HALYARD* CHEMOTHERAPY PROCEDURE GOWN

- Potential for soaking and heavy fluid contact in front
- Tested against 52 chemotherapy drugs
- Meets Liquid Barrier Performance Standards ASTM 1670 and 1671
- Highest level of protection against flammability
- Polypropylene with polyethylene coating



Code	Description	Packaging
69606	Procedure Gown for Chemotherapy Use, Over-the-head Open-back, Poly-coated, Universal	Eaches/Case = 100
37284	Procedure Gown for Chemotherapy Use, Over-the-head Open-back, Poly-coated, XX-Large	Eaches/Case = 100



HALYARD* CHEMO360* GOWN

- Closed back chemotherapy gown
- Potential for soaking and heavy fluid contact in front
- Tested against 52 chemotherapy drugs
- Meets Liquid Barrier Performance Standards ASTM 1670 and 1671
- Highest level of protection against flammability
- Polypropylene with polyethylene coating



Code	Description	Packaging
47346	CHEMO360* Gown, Closed-back, Poly-coated, Tape closure, Universal	Eaches/Case = 100
47347	CHEMO360* Gown, Closed-back, Poly-coated, Tape closure, XX-Large	Eaches/Case = 100

NOTE: Please see package inserts for directions for use for each of these products.
Always read the label and follow the directions for use.

PROTECTIVE APPAREL

SCRUB SUITS

HALYARD* SCRUB SHIRTS



- Potential for light fluid contact
- Protective tri-layer SMS fabric
- Scoop neck
- Reinforced pocket

Code	Description	Packaging
69701	HALYARD* Scrub Shirt, Tri-Layer Fabric, Blue, Medium	Eaches/Case = 48
69702	HALYARD* Scrub Shirt, Tri-Layer Fabric, Blue, Large	Eaches/Case = 48
69703	HALYARD* Scrub Shirt, Tri-Layer Fabric, Blue, X-Large	Eaches/Case = 48

HALYARD* SCRUB PANTS

- Potential for light fluid contact
- Protective tri-layer SMS fabric
- Drawstring waist
- Side and rear pockets
- Reinforced in-seam

Code	Description	Packaging
69711	HALYARD* Scrub Pants, Blue, Medium,	Eaches/Case = 48
69712	HALYARD* Scrub Pants, Blue, Large	Eaches/Case = 48
69713	HALYARD* Scrub Pants, Blue, X-Large	Eaches/Case = 48

NOTE: Please see package inserts for directions for use for each of these products
Always read the label and follow the directions for use..

PROTECTIVE APPAREL

HEADWEAR

HALYARD* BOUFFANT CAPS

- No potential fluid contact
- 1-Layer spunbond fabric
- Regular size



Code	Description	Packaging
3501	Bouffant Cap, Spunbond Fabric, Blue	Eaches/Dispenser = 100 Dispensers/Case = 5 Eaches/Case = 500
3502	Bouffant Cap, Spunbond Fabric, White	Eaches/Dispenser = 100 Dispensers/Case = 5 Eaches/Case = 500



HALYARD* SURGICAL CAP

- For light fluid contact
- Tri-Layer SMS fabric on sides
- Spunbond fabric on top
- Universal size

Code	Description	Packaging
69520	Surgical Cap, Blue	Eaches/Dispenser = 100 Dispensers/Case = 3 Eaches/Case = 300



HALYARD* SURGICAL HOOD

- For light fluid contact
- Tri-layer SMS fabric on sides
- Spunbond fabric on top
- Tie neck
- Universal size

Code	Description	Packaging
69110	Protective Surgical Hood, Blue	Eaches/Dispenser = 100 Dispensers/Case = 3 Eaches/Case = 300

NOTE: Please see package inserts for directions for use for each of these products.
Always read the label and follow the directions for use.

PROTECTIVE APPAREL

FOOTWEAR

X-TRA TRACTION* SHOE COVERS

- Blue
- Protective tri-layer SMS fabric
- Traction strips
- Natural rubber latex-free



Code	Description	Packaging
69252	X-TRA TRACTION* Shoe Cover, Universal	Eaches/Dispenser = 100 Dispensers/Case = 3 Eaches/Case = 300
69254	X-TRA TRACTION* Shoe Cover, X-Large	Eaches/Dispenser = 80 Dispensers/Case = 3 Eaches/Case = 240

ANKLE TOPS

ANKLE GUARD* SHOE COVER

- Moderate fluid contamination
- Over the ankle coverage
- Protective tri-layer SMS fabric
- Adhesive traction strips
- Natural rubber latex-free



Code	Description	Packaging
69353	ANKLE GUARD* Shoe Cover, Tri-Layer Fabric, Adhesive Traction Strips, X-Large	Eaches/Dispenser = 90 Dispensers/Case = 3 Eaches/Case = 270

KNEE HIGHS

HI-GUARD* ULTRA FULL-COVERAGE BOOT

- Potential for heavy fluid contamination
- Knee high boot
- Plastic film coating over tri-layer fabric
- Two 1", 2.5cm adhesive foam traction strips
- Natural rubber latex-free



Code	Description	Packaging
69572	HI-GUARD* Full Coverage Boot, Universal	Eaches/Dispenser = 30 Dispensers/Case = 4 Eaches/Case = 120
69672	HI-GUARD* Full Coverage Boot, X-Large	Eaches/Dispenser = 30 Dispensers/Case = 4 Eaches/Case = 120

NOTE: Please see package inserts for directions for use for each of these products.
Always read the label and follow the directions for use.



Infection Prevention Product Solutions
Halyard* Education Foundation
Tools & Best Practice
Commitment to Excellence and Sustainability
Experienced Customer Support
Knowledgeable Sales Force
Innovation & Technology



O&M Halyard Australia Pty. Limited

AUS: 1800 664 227

www.halyardhealth.com.au

www.halyardhealth.com.au



Halyard makes no representation that its products described in this catalogue will provide complete protection from exposure to contaminants, fluids or micro-organisms or will eliminate the risk of contracting infectious diseases.

Registered Trademark or Trademark of O&M Halyard or its affiliates. ©2022 O&M Halyard, Inc.
All Rights Reserved. 2022:06:WFHY947v6