



# SEQUENCE FOR DONNING PERSONAL PROTECTIVE EQUIPMENT (PPE):

## 1. PROTECT YOUR BODY

Ensure the gown:

- Covers torso from neck to knees
- Covers arms to end of wrists
- Wraps around the sides to fasten at neck and back



## 2. PROTECT YOUR FACE

- Create a curve in the flexible band for the nose bridge and expand the mask
- Secure the ties or elastics at the middle of the head and neck, or fasten the ear loops
- Fit it snugly to your face and below your chin – fit check the N95 and P2 mask
- Place the face shield or eyewear over the face and eyes and adjust to fit



## 3. PROTECT YOUR HANDS

- Extend the cuffs to cover the wrist of the gown
- Limit the surfaces touched
- Change gloves when torn or heavily contaminated
- Perform hand hygiene as required



## FINALLY STOP and CHECK all PPE is in place



# SEQUENCE FOR ADVANCED DOFFING PERSONAL PROTECTIVE EQUIPMENT (PPE):

Remember! The outside and front of your PPE, including the sleeves of the gown, are contaminated. If hands become visibly contaminated during PPE removal, wash hands before continuing.

## REMOVE GOWN THEN GLOVES

Outside of gown and gloves are **CONTAMINATED!**

- 1a.**
- With gloved hands, grasp the front of the gown at the shoulders and pull the gown forward to snap the tabs at the back
  - Peel off the gown, rolling it inside out
- 1b.**
- Continue over the gloves so that they too are rolled inside out
  - Discard gown and gloves into a waste container
  - Perform hand hygiene using ABHR or soap and water



## 2. REMOVE PROTECTIVE EYEWEAR

- Grasp the ear or head piece and lift away from face
- Place reusable eyewear in the designated receptacle for cleaning or
- Discard disposable eyewear into waste container for disposal



## 3. REMOVE MASK OR PARTICULATE RESPIRATOR (N95 OR P2)

- Remove by touching ties or elastics only, starting with the bottom then top ties or bands
- Discard in designated waste container



## 4. PERFORM HAND HYGIENE

- Wash hands with soap and water or use an alcohol-based hand rub immediately after removing PPE



# PROTECTIVE APPAREL



## HALYARD\* ISOLATION GOWN

- For use when expected risk of exposure to fluid is between low and moderate
- Medium-weight tri-layer fabric
- Tape-tab neck closure
- Tie waist, elastic cuffs

<b>Code</b>	<b>Description</b>	<b>Packaging</b>
69979	Isolation Gown, Yellow, Large	Eaches/Bundle = 10 Bundles/Case = 10 Eaches/Case = 100

## HALYARD\* BASICS\* TRI-LAYER AAMI 2 OVER-THE-HEAD ISOLATION GOWN WITH THUMBHOOKS

- Meets AAMI Level 2 requirements
- Medium-weight Tri-layer fabric
- Full back coverage with waist tie for easier closure
- Reinforced ultrasonic seam



<b>Code</b>	<b>Description</b>	<b>Packaging</b>
43008	Thumbhooks, Yellow, Closed Back, extended front panel, Large	Eaches/Bundle = 10 Bundles/Case = 10 Eaches/Case = 100
43009	Thumbhooks, Yellow, Closed Back, extended front panel, X-Large	Eaches/Bundle = 10 Bundles/Case = 10 Eaches/Case = 100

NOTE: Please see package inserts for directions for use for each of these products. Always read the label and follow the directions for use.

# PROTECTIVE APPAREL



69455

## HALYARD\* IMPERVIOUS COMFORT GOWNS

- Potential for moderate to heavy fluid contact in front
- Tri-layer spunbond with polyethylene laminate coating
- Breathable spunbond back
- Knit cuffs
- Velcro neck tabs
- Tie waist

Code	Description	Packaging
69455	Impervious Open Back Gown, Knit Cuff, Blue, Large	Eaches/Bundle = 30 Bundles/Case = 2 Eaches/Case = 60
5511	Impervious Full Back Comfort Gown, Yellow, Regular	Eaches/Case = 50
5512	Impervious Full Back Comfort Gown, Yellow, X-Large	Eaches/Case = 50



5511

## HALYARD\* IMPERVIOUS THUMBS-UP\* FILM GOWNS

- Potential for soaking and heavy fluid contact in front
- Impervious polyethylene film
- Open back, thumbhooks

Code	Description	Packaging
7000	Impervious Thumbs-Up* Film Gown with Thumbhooks, Blue, Regular	Eaches/Dispenser = 15 Dispensers/Case = 5 Eaches/Case = 75
7001	Impervious Thumbs-Up* Film Gown with Thumbhooks, Blue, X-Large	Eaches/Dispenser = 15 Dispensers/Case = 5 Eaches/Case = 75
7002	Impervious Thumbs-Up* Film Gown with Thumbhooks, Yellow, Regular	Eaches/Dispenser = 15 Dispensers/Case = 5 Eaches/Case = 75
7003	Impervious Thumbs-Up* Film Gown with Thumbhooks, Yellow, X-Large	Eaches/Dispenser = 15 Dispensers/Case = 5 Eaches/Case = 75



7000



7002

NOTE: Please see package inserts for directions for use for each of these products. Always read the label and follow the directions for use.

# PROTECTIVE APPAREL



## HALYARD\* CHEMOTHERAPY PROCEDURE GOWN

- Potential for soaking and heavy fluid contact in front
- Tested against 52 chemotherapy drugs
- Meets Liquid Barrier Performance Standards ASTM 1670 and 1671
- Highest level of protection against flammability
- Polypropylene with polyethylene coating



Code	Description	Packaging
69606	Procedure Gown for Chemotherapy Use, Over-the-head Open-back, Poly-coated, Universal	Eaches/Case = 100
37284	Procedure Gown for Chemotherapy Use, Over-the-head Open-back, Poly-coated, XX-Large	Eaches/Case = 100



## HALYARD\* CHEMO360\* PROCEDURE GOWN

- Closed back chemotherapy gown
- Potential for soaking and heavy fluid contact in front
- Tested against 52 chemotherapy drugs
- Meets Liquid Barrier Performance Standards ASTM 1670 and 1671
- Highest level of protection against flammability
- Polypropylene with polyethylene coating

Code	Description	Packaging
47346	CHEMO360* Gown, Closed Back, Poly-Coated, Tape Closure, Blue, Universal	Eaches/Case = 100
47347	CHEMO360* Gown, Closed Back, Poly-Coated, Tape Closure, Blue, XX-Large	Eaches/Case = 100

NOTE: Please see package inserts for directions for use for each of these products. Always read the label and follow the directions for use.

# PROTECTIVE APPAREL

## SCRUB SUITS

### HALYARD\* SCRUB SHIRTS



- Potential for light fluid contact
- Protective tri-layer SMS fabric
- Scoop neck
- Reinforced pocket

<b>Code</b>	<b>Description</b>	<b>Packaging</b>
69701	HALYARD* Scrub Shirt, Tri-Layer Fabric, Blue, Medium	Eaches/Case = 48
69702	HALYARD* Scrub Shirt, Tri-Layer Fabric, Blue, Large	Eaches/Case = 48
69703	HALYARD* Scrub Shirt, Tri-Layer Fabric, Blue, X-Large	Eaches/Case = 48

### HALYARD\* SCRUB PANTS

- Potential for light fluid contact
- Protective tri-layer SMS fabric
- Drawstring waist
- Side and rear pockets
- Reinforced in-seam

<b>Code</b>	<b>Description</b>	<b>Packaging</b>
69711	HALYARD* Scrub Pants, Blue, Medium,	Eaches/Case = 48
69712	HALYARD* Scrub Pants, Blue, Large	Eaches/Case = 48
69713	HALYARD* Scrub Pants, Blue, X-Large	Eaches/Case = 48

NOTE: Please see package inserts for directions for use for each of these products.  
Always read the label and follow the directions for use.

# PROTECTIVE APPAREL

## HEADWEAR

### SURGICAL CAP



- For light fluid contact
- Tri-Layer SMS fabric on sides
- Spunbond fabric on top
- Universal size

<b>Code</b>	<b>Description</b>	<b>Packaging</b>
69520	Surgical Cap, Blue	Eaches/Dispenser = 100 Dispensers/Case = 3 Eaches/Case = 300

### SURGICAL HOOD



- For light fluid contact
- Tri-layer SMS fabric on sides
- Spunbond fabric on top
- Tie neck
- Universal size

<b>Code</b>	<b>Description</b>	<b>Packaging</b>
69110	Protective Surgical Hood, Blue	Eaches/Dispenser = 100 Dispensers/Case = 3 Eaches/Case = 300

NOTE: Please see package inserts for directions for use for each of these products. Always read the label and follow the directions for use.